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The following precautionary articles show how you operate our products safe and correct, which prevents you from any danger or damage before happened.

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Never fail obeying to any article, in which crucial and important matters are shown concerning safety.

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This symbol indicates the possibility of being dead or wounded seriously if our products are operated by mistake regardless of this instruction.

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This symbol indicates the possibility of being hurt or generating material damages if our products are operated by mistake regardless of this instruction.

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Please work in due order. Knitted gloves, clothes with open cuffs and long hairs are in danger of being caught in a machine.

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Please put goggles on not to enter dusts and grits into eyes. In addition, please put a protection mask on not to inhale dusts and grits.

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Please use Appliances in a place where the temperature is from 0 to 40 °C and the relative humidity is from 20 to 80% (no dew condensation). Please never use them, either in humid, wet places and outdoors in the rain. It may invite electric shock or fire, weaken insulation and cause rust and bad operation.

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Do not use absolutely in any place where there are flammable or explosive materials such as lacquer, paints, thinner, gasoline, gas and adhesives.

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Also, ensure that there is no flammable and no corrosive gas areas.

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Please do not damage the AC cord and the Motor unit cord to prevent electric leakage and shock or fire.

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Please use tools such as bur and point with specified standard within the permitted range of rotation.

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Please work in a clean and full space and use Appliances in well orderly areas not to stumble over cords.

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Please do not place a cloth or anything else on the ventilation slits of Appliance. Also, please avoid using while any dust is attached around the ventilation outlet.

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Please take care not to allow any alien substances such as water, oil, and dust into Appliances.

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Please do not blow something like compressed air into ventilation inlet of Appliance.

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Please keep Appliances in a place not to be reached by Children.

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Please never make any modification, which may damage original performance and safety.

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Please keep Appliances in well dried areas.

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Do not operate under the off-balance.

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Never fail operating for any application not appointed.

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Please never fail using Controller under the voltage indicated in the panel.

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While the handpiece is operating, keep hands, eyes and face away from all rotating parts.

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Please keep Appliances in well dried areas in a place not to be reached by Children.

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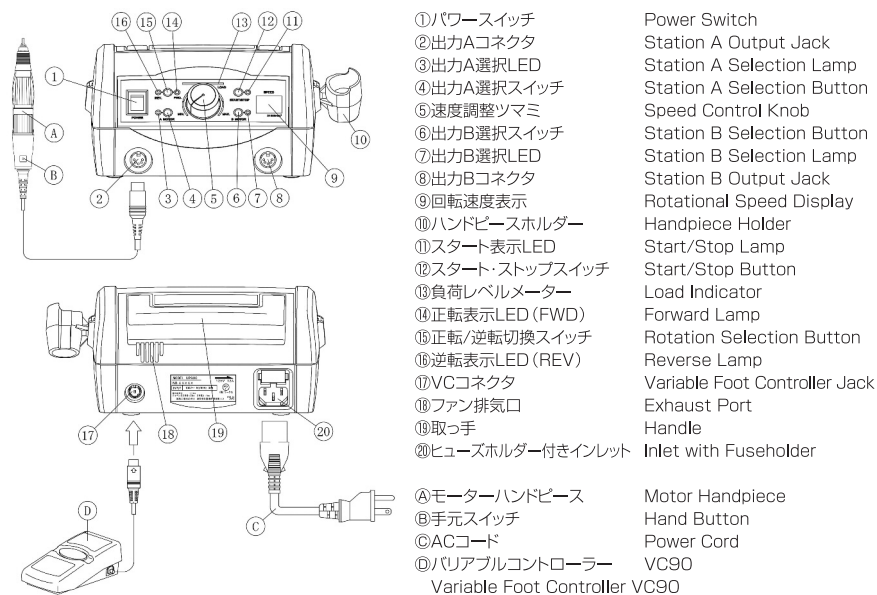
After use, make sure there is nothing unusual in the equipment providing for next use. If something unusual is detected, do not leave it as it is, and contact your supplier.

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Periodical inspection is recommended every 1 or 2 years to use the appliance safely and efficiently. Please consult your supplier for the service.

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(1) Ensure that Power Switch① is turned off before connecting Power CordⒸ to Inlet ⑳ and to the power supply.

(2) Connect the Motor Handpiece to Station A② or Station B Output Jack.⑧

(3) If required, connect the Variable Foot Controller VC90⑨ to Variable Foot Controller Jack.⑰

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- (1) Set the Speed Control Knob⑤. at 「MIN.」 position.
- (2) Turn on the Power Switch①. Then, Lamp that shows the selective state of a present direction of the rotation and the output jack lights. Moreover, if the Motor Handpiece is connected with the selected output jack, the set speed is displayed in the Rotational Speed Display⑨.
- (3) Push the Rotation Selection Button⑮ momentarily to set forward or reverse state.
- (4) The output jack can be changed with Station A Selection Button④ or Station B Selection Button⑥.
- (5) Please set the rotational speed beforehand with the Speed Control Knob⑤.
- (6) When Start/Stop Button⑳ or Hand Button㉑ is pushed, the Motor Handpiece rotates.
- (7) When Start/Stop Button⑳ or Hand Button㉑ is pushed again, the Motor Handpiece stops.

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You can use the Variable Foot Controller VC90㉑ to vary the speed of the Motor Handpiece too, by adjusting the pressure applied to the Variable Foot Controller. In this case, the maximum speed is limited by the Speed Control Knob⑤.

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The rotational speed can be maintained by pushing the Start/Stop Button⑳ or Hand Button㉑ to fix the speed while using the Foot Controller. In that case, Start/Stop Lamp⑪ blinks. Please push again Start/Stop Button⑳ or Hand Button㉑ to release it or step on the pedal once.

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When you do not use the Motor Handpiece for a while, the oxide layer might be formed to the motor commutator, and it not rotate in low speed. In that case, please remove the oxide layer by driving at high speed for tenth of seconds.

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Load Indicator⑬ is composed of 8 lamps. The number of lighting increases as the load of the motor rises.

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Please work within the range where Green of the Load Indicator⑬ lights.

When working in high pressure (Red) for a long time, Circuit Breaker works for safety.

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While handpiece is rotating without any pressure, if two green lamps or more of the Load Indicator⑬ light up, there is a possibility that some part of handpiece is diffective. Please consult your supplier.

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Rotational Speed Display⑨ is displayed as follows.

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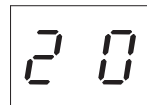
Circuit breaker works, and it will temporarily stops power supply when operated abnormally such as too much load, handpiece is locked, turned set ring by mistake while rotating and etc...

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When the handpiece becomes an overloaded status, warning is sounded.

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When circuit breaker works, handpiece stops, and all of load indicator lights up and 「OL」 is displayed at digital display.

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Please push Start/Stop Button⑫, Hand Button⑩, Station A Selection, Button④ or Station B Selection Button⑥ to release the Circuit Breaker.

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When the Variable Foot Controller is used, it is possible to release it by returning the pedal to original position.

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When it is used in high pressure for a long time, internal temperature of the controller will be high, and Thermal Sensor will automatically stop electric transmission temporarily.

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When thermal sensor works, the Handpiece stops and 'OH' is displayed at digital display.

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When the internal temperature falls, thermal sensor is automatically released.

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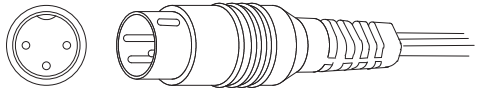
An internal cooling fan operates while overheat. Please wait for a while without turning off Power Switch①

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Old Handpieces with 3 pin plug as the figure above, it is possible to connect to G7 controller.

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It is possible to use our conventional handpieces. However, please select output jack by yourself. The rotational speed display becomes like mentioned before (5. ROTATIONAL SPEED DISPLAY).

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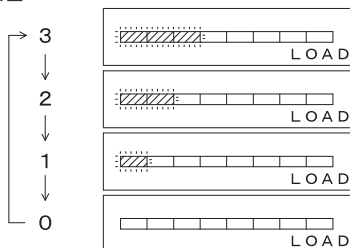
When you turn on Power Switch① while pushing Start/Stop Button⑫, 'SP' is displayed and Load Indicator⑬ lights according to present setting. The volume can be set as LED of the Load Indicator⑬ repeats '3→2→1→ all turning off' whenever Start/Stop Button⑫ is pushed. After setting it, it becomes usual operation by turning on Power Switch① again.

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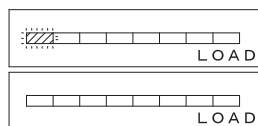
有効  
Effective  
↕  
無効  
No Effect


When the operation with Hand Button⑥ is unnecessary, Hand Button⑥ can be invalidated. When you turn on Power Switch① while pushing Station B Selection Button⑥, 'HS' is displayed and Load Indicator⑬ lights according to a present setting. Effective and No Effect in Hand Button⑥ are switched as LED of Load Indicator⑬ is lit and turned off. After setting it, it becomes usual operation by turning on Power Switch① again.

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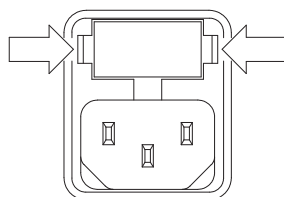
There are fuses in Inlet with Fuseholder⑳. Fuses can be changed by removing a lock on both side, and pulling out the fuseholder. Please use connect fuse according to the plug.

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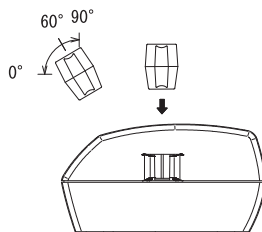
Handpiece Holder⑩ is attachable with angle  
60° or 90°.

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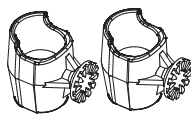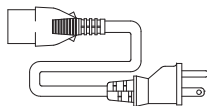
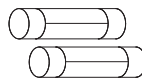
ヒューズ 2本  
Fuse 2 pcs  
(100V・200V UF-2A-1)  
(230V UF-0.8A-1)

## EC Declaration of Conformity

Application of Council Directive : 2006/42/EC、2004/108/EC

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\* The specifications and designs in this Instruction Manual are subject to change without notice.

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The trade mark of 「」is registered officially by Urawa Corporation.

IM90-409 1106